
Extremity Positioning Tables

EXPO EXPO-II

OPERATOR'S MANUAL



**HCMI, Inc.
2146 East Pythian
Springfield, MO 65802
Phone (417) 864-6511
Fax (417) 864-7394**



TABLE OF CONTENTS Expo/Expo-II Extremity Positioning Tables

Introduction	Page 2
Purpose of the Expo/Expo-II	Page 2
Limited Warranty	Page 3-4
Treatment Disclaimers	Page 4
Product Information	Page 5
Table Specifications	Page 5
Installation Procedures	Page 6
Uncrating Procedure	Page 6
Control Functions	Page 7
Parts Replacement	Page 8
Instructions for Damaged Merchandise	Page 8
Resale or Disposal	Page 8



1: INTRODUCTION

Your new **HCMI, Inc. Expo/Expo-II Extremity Positioning Tables** have been designed and engineered to comply with the rigid standards of quality so necessary to successful practitioners worldwide.

PURPOSE OF THE EXPO

The **EXPO** extremity positioning table is the first table designed for chiropractic radiography. Small enough to sit alongside an x-ray tubestand, the **EXPO** is the most innovative table available today. Designed for extremity radiography, the **EXPO** utilizes two drop leaf sections. Either section or both can be quickly and securely fastened to meet every x-ray application. With the drop leafs down, the **EXPO** table allows for full vertical travel at the tubestand's 40" S.I.D.

PURPOSE OF THE EXPO-II

The **EXPO-II** offers chiropractors the ability to quickly add another treatment table to the office. Expanding to a full 72 inches, the **EXPO-II** offers standard features of a seamless face slot, padded arm rests, and locking casters.

Your new instrument is designed to provide you with years of service. With proper maintenance, your table will remain virtually trouble-free.



2: IMPORTANT SAFETY INSTRUCTIONS

Subject to the exceptions and upon the conditions stated below, **HCMI, Inc.** warrants that the **Expo/Expo-II Extremity Positioning Tables** and its options, will be free from defects in material and construction for a period of one (1) year, used or not, to the original purchaser, after the date of sale. **HCMI, Inc.** agrees that if any such product should prove to be defective within said warranty period, it will replace any defective part for a said warranty period. This warranty applies only if determination is made that the defect developed under normal and proper use.

The purchaser agrees to pay for all labor and/or parts determined to be not warranted and to pay all expenses accrued in doing said repairs.

It is expressly agreed that the above warranty will be in lieu of all warranties of fitness and of the warranty of merchantability and that **HCMI, Inc.** will have no liability for special or consequential damages of any kind or from any cause at all arising out of manufacture, use, sale, handling, repair, maintenance, or replacement of any of the products sold under this warranty.

Representation and warranties formed by any person, including dealers and representatives of **HCMI, Inc.**, which are inconsistent or in conflict with the terms of this warranty, will not be binding upon **HCMI, Inc.** unless reduced to writing and approved by the Board of Directors of **HCMI, Inc.**



2: LIMITED WARRANTY (continued)

Buyers will notify **HCMI, Inc.** in writing by registered mail also certified mail or return receipt requested, **within thirty (30) days of a breach of guarantee after discovery thereof; otherwise, such claims will be deemed to be waived.**

TREATMENT DISCLAIMER

No product or device can be defective in whole or in part because of biological differences among individuals. Therefore, **HCMI, Inc.** does not represent or warrant under any circumstances that a particular result can or will be achieved by using its product or products. The choice of use of the product, the method or methods of use, the diagnosis of the patient's condition and the application for the patient is solely the responsibility of the practitioner and not of **HCMI, Inc.**



3: PRODUCT INFORMATION

EXPO SPECIFICATIONS:

Base length: 42"
Full extended length: 65"
Width: 23"
Height: SPECIFY 22"-30"

EXPO-II SPECIFICATIONS:


Base length: 42"
Full extended length: 72"
Width: 23"
Height: SPECIFY 22"-30"



4: INSTALLATION PROCEDURES

UNCRATING PROCEDURE

1. Remove all exterior crating material.

 **Caution:** Table covering is very susceptible to tearing. Use caution when using sharp tools!

2. Carefully lift the table out of the particle board.



5: CONTROL FUNCTIONS

To relax **Cushion Assembly**, pull out on locking arm, found under cushion. To bring cushion to prone position, lift cushion and slide locking arm into locking plate.

Optional: **301 Face Slot and Hand Rest**

Optional: **Locking Casters**-Push down on lever to lock and lift up to release.



6: PARTS REPLACEMENT

NOTE: (Refer to Warranty Information in this manual.)

When replacing worn parts, use genuine **HCMI, Inc.** replacement parts by contacting our service department. When ordering replacement parts:

- ◆ Ask for the service department.
- ◆ Have the model/serial number of the unit ready to give to the service representative.
- ◆ Specify the part by number/description.

We are here to help if a problem should arise. Always feel free to contact our service representative for assistance at **(800)-641-4107**.

INSTRUCTIONS FOR DAMAGED MERCHANDISE

The contents of this shipment have been checked and packed by experienced personnel. Articles damaged in transit should be reported at once to the delivering carrier and a claim must be filed by you. We can neither be responsible for, nor can we accept the return of merchandise damaged in transit.

RESALE OR DISPOSAL

A chiropractic table is considered a medical device by the Food and Drug Administration. Therefore, it is necessary that **HCMI, Inc.** be notified if your table is sold, destroyed, or otherwise disposed of.

Please notify in writing to:

HCMI, INC
2146 EAST PYTHIAN
SPRINGFIELD, MO 65802

Please include the unit serial number, the person that the table was sold to, or the reason for disposal. Thank you for your cooperation.